



Validated European Cleaning instructions for

Easel and Slider

1. Place item in pouch and seal pouch.
2. Place pouch on a metal tray into the autoclave
3. When instruments contain any type of plastic, place a towel on the tray first.
4. Follow the required autoclave settings below.

Minimal exposure times for steam sterilisation:

Temperature	Pressure	Time
121° C	15 psi	15 minutes
126°C	20 psi	10 minutes
134°C	29.4 psi	3 minutes.

When autoclave cycle is complete, remove item from the pouch and place directly into the sterile field.