



## 6 STEPS, 6 MINUTES ➤ READY TO BE COLLECTED!

**2**

**N.B.**

Step 2 is only required for the first cycle of processing.

Should you wish to process further cycles it is not necessary to prime the filter again.

- \* Attach the I.V. line to the WASH IN valve on the device and open the clip to allow approx 100 - 150 ml of saline solution into the bag to moisten the filters. Tip the Beauty-Stem vertically for a moment to allow the fluid to remove any air.

- \* Then close the clamp on the I.V. line again.


- \* Open the I.V. line to begin the continuous washing process.
- \* And open the waste tube clamp to eliminate residues.

**6**

- \* Retrieve the adipose tissue from the LIPO OUT valve.

**4**

I.V. line

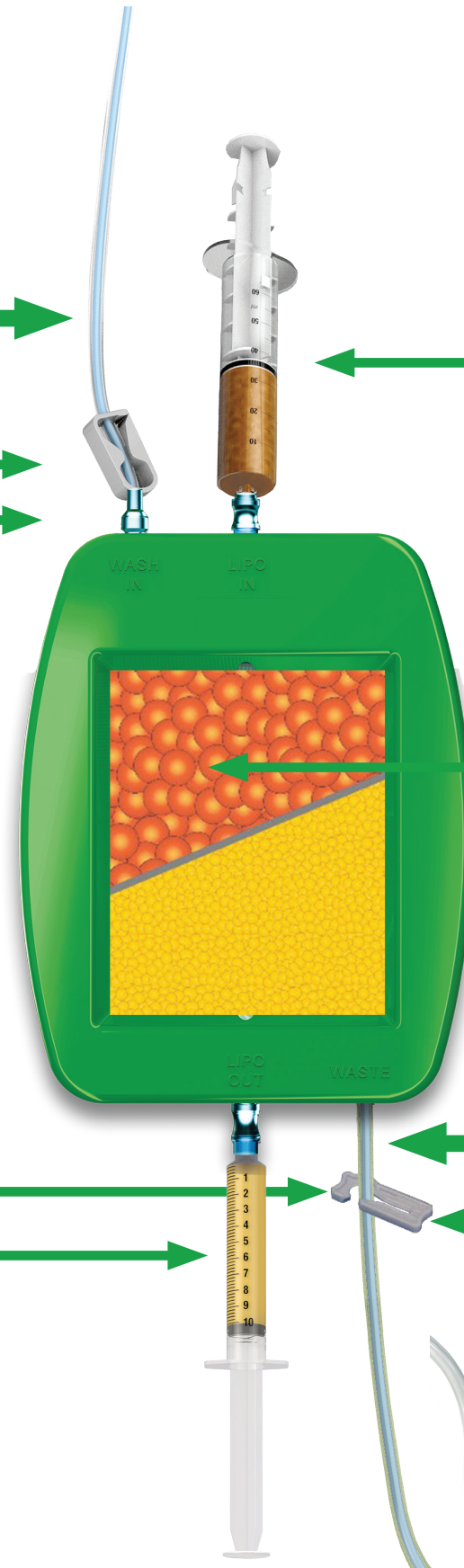

**3**

- \* Connect the lipoaspirate syringe to the LIPO IN valve
- \* Inject the fat into the device. Approx 200cc of lipoaspirate can be processed in each cycle.

**5**

- \* Gently move the spatula up and down the filter bag until the fat tissue turns yellow and the liquid in the waste tube becomes transparent.
- \* Close the I.V. line clip at the WASH IN valve, and continue to use the spatula several more times to remove any remaining saline solution

**1**

- \* Close the blue tap on the waste bag
- \* Close the clamp on the waste tube



SURGICAL TECHNIQUE:  
WATCH THE VIDEO



SURGICAL TECHNIQUE:  
READ THE GUIDE